

Customer Name: Georgia Mountain Health  
 Reference: Super Friendly Dental Preference using  
 Quote #: existing EQ  
 WQBEV000955-2



**Technical Document**

Line	Item	Label
5	5730	Left Hinge
6	5730	Right Hinge
14	5730	Left Hinge

**5730 Dispenser Unit or 5731 Storage Unit**

**WARNING:** For 22" or 24" models (22-5-31 (5730) unit or 24-5-31 (5731) unit) model, you must mount the module on a design specified to hold it to its base location (see Figure 12).

For 30" or 42" models (30-5-31 (5730) unit or 42-5-31 (5731) unit) model, you must mount the module on a design specified to hold it to its base location (see Figure 12). If the ends of the machine slip, see note 10" (24" units from a dual stud attachment, you must install an additional toggle bolt and/or attach the module to the gypsum wallboard. Install the additional toggle bolt within approximately 10" (24" units) from the dual stud attachment point.

If you cannot make the proper number of stud attachments, you must change the stud spacing or reduce the wall bearing. Refer to page 2 for requested conditions.

**Metal Studs**

1-1/2" x 22 g.n. stud  
 Top: 22" (568.8 mm), 24" (609.6 mm), 30" (762.0 mm), and 42" (1066.8 mm)  
 15W, anchors to stud flange  
 Bolts: #4 x 2-1/2" x 5.0L screws in each stud

**Wood Studs**

2x4 or 2x6 or D.F. or S.P.  
 Top: #10 x 8" wood screws  
 Bolts: #10 x 3" wood screws in each stud

This written material is not intended to be, and should not be, used as a substitute for the compliance with local, state, federal code and Americans with Disabilities Act. It is the responsibility of the user to integrate the A-dec equipment specifications presented herein with the regulations and codes overseen by local planning and inspection authorities. A-dec, Inc., shall not be held liable for any errors contained herein or any consequential or other damages concerning the furnishings, performance or use of this material. The information in this document is subject to change without notice. If you find any problems with this document, please report them to us in writing.

Order Plans @ www.LDILine.com



PROJECT: GEORGIA MOUNTAINS HEALTH

LOCATION: ELLIJAY, GA.

HENRY SCHEIN REP: JOE WILSON

CENTER: ATLANTA, GA.

PHONE #: (404)-226-8821

**-IMPORTANT NOTE-**  
 THIS DOCUMENT HAS BEEN PREPARED AS AN INSTRUMENT OF PROFESSIONAL SERVICE AND IS NOT AN ARCHITECTURAL PLAN. THE IDEAS/ DESIGN INCLUDED HEREIN ARE THE SOLE PROPERTY OF HENRY SCHEIN DENTAL AND ARE PROTECTED UNDER COPYRIGHT. THESE PLANS MAY NOT BE USED OR REPRODUCED WITHOUT EXPRESSED WRITTEN CONSENT OF HENRY SCHEIN AND FULL PAYMENT OF ANY ASSOCIATED DESIGN FEES.  
**ALL DIMENSIONS ARE SUBJECT TO JOB-SITE VERIFICATION.**

PROJECT NUMBER: 20-0565

PROJECT START DATE: 4/3/2020

FINALS START DATE: 7/9/2020

DRAWN BY: ARCH  
 FINALS BY: TJK  
 CHECKED BY: LBS

REVISIONS:

---/---/---	---
---/---/---	---
---/---/---	---
---/---/---	---
---/---/---	---
---/---/---	---
---/---/---	---
---/---/---	---

INT.SQ.FT.= PER ARCH

SCALE: N/A SH. SIZE: 24 x 36

DENTAL CABINET DRAWINGS

DDC.6